

## Comfor™ Shoulder Immobilizer – Universal



Figure 1



Figure 2



Figure 3



Figure 4

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

**Intended for single patient use.**



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Unfold swathe (A) and position contoured portion on front mid-section. Wrap swathe around waist and secure (Figure 1).
2. Position humeral strap (B) under the affected arm just above the elbow and attach to swathe with narrow hook strip (C) (Figure 2). Wrap humeral strap (B) around affected arm and secure to back of swathe (Figure 3).
3. Secure the end of shoulder strap (D) without a buckle to the back of the swathe (Figure 3).
4. Pull strap (D) over unaffected shoulder and secure to the front of swathe. Use slide buckle to adjust length of shoulder strap (Figure 2).
5. Wrap and secure cuff (E) on wrist of affected arm. Secure cuff to chest swathe (Figure 4).

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product and warranty information or contact a customer care representative at **800-328-1095**.

**NOTICE:** This device is only one element in the overall treatment program administered by a medical professional. While every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

**NOT MADE WITH  
natural rubber latex**